



6256 34<sup>th</sup> Avenue NW  
 Rochester MN 55901  
 1.507.289.0761 Phone  
 1.507.281.9096 Fax  
 Medicalinnovations.com



## BARTON - MAYO™ TRACHEOSTOMA BUTTON™

### PATIENT SIZING KIT

Fitting of the Barton-Mayo™ Tracheostoma Button to the individual patient's stoma is critical. Proper sealing of the button is achieved by correctly positioning the inner lip of the button in the stoma. Button size is based on stoma diameter, depth, and angle. Often, many buttons must be tried in order to obtain the best fit.

To facilitate the sizing procedure, a sizing kit is available which includes trial buttons of each Barton-Mayo™ button available. After a patient fitting session, simply clean and steam autoclave the trial buttons to reuse. To promptly receive a useable button for the patient, complete the prescription form and mail to Medical Innovations International, Inc.

### Sizing Chart

<u>Size</u>	<u>Descriptive</u>	<u>Dimensions (mm)</u>		<u>Catalog</u>
<u>Number</u>	<u>Length</u>	<u>Diameter</u>	<u>Length</u>	<u>Number</u>
9	Short	13.5	15	EN89900-091
9	Regular	13.5	22	EN89900-092
9	Long	13.5	30	EN89900-093
10	Short	15.0	15	EN89900-101
10	Regular	15.0	22	EN89900-102
10	Long	15.0	30	EN89900-103
12	Short	17.0	15	EN89900-121
12	Regular	17.0	22	EN89900-122
12	Long	17.0	30	EN89900-123
14	Short	19.0	15	EN89900-141
14	Regular	19.0	22	EN89900-142
14	Long	19.0	30	EN89900-143

**Barton-Mayo™ Tracheostoma Button Sizing Kit order number EN89910-000**

Individual buttons can be ordered directly to maintain a supply of buttons, or purchased by individuals with the prescription form for specific patients (prescription form available on the company website).